




# CathGrip®

## Low Profile Double Strap

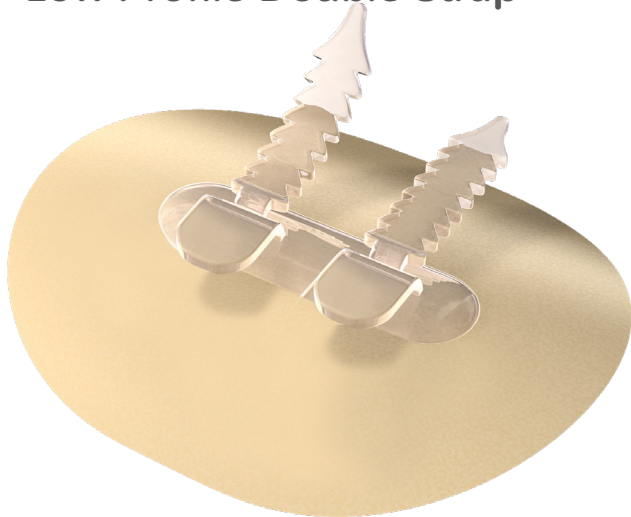

FOR INSTITUTIONAL USE ONLY

### Contraindications:

#### Absolute (When the product should not be used at all)

- Demonstrated sensitivity to hydrocolloid (rare)

#### Relative (When the product can be used after healing)

- Unhealed wound at application site (wait to heal)
- Active rash at application site (wait to heal)
- Active inflammation or infection

#### For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

Any serious incident while using this device should be reported to the manufacturer and competent authority.


NOT MADE  
WITH  
NATURAL  
RUBBER  
LATEX



Distributed by:  
Medline Industries, LP  
Northfield, IL 60093, USA  
Support: (800) 633 5463  
CustomerService@Medline.com  
www.Medline.com



Manufactured by:  
BioDerm, Inc.  
12320 73<sup>rd</sup> Court North, Largo, FL 33773, USA  
Support: (800) 864-6755  
CustomerCare@BioDermInc.com  
www.BioDermInc.com

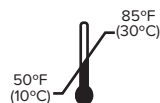

## Application Tips

Make sure the patient's skin is **COMPLETELY DRY** before applying CathGrip®.

Cleansing wipes and Chlorhexidine can cause excess moisture underneath the skin that can reduce CathGrip wear time. Make certain that the application site for CathGrip is as dry and moisture free as possible, before applying.

**Wear time.** The first application for each patient will last shorter than the subsequent applications of the product. This is due to the hydrocolloid material that absorbs underlying moisture in the skin. CathGrip can be worn for up to 7 days.

**Single-Use.** Product is not reusable and could cause product malfunction or risk of infection if reused. Once removed, dispose of in a trash can.

## Removing CathGrip

FreeDerm Instantly removes CathGrip with a gentle, quick drying, no washing, no-sting, skin friendly formula. Remove and replace CathGrip when the adhesive turns a white color, when it is loose, or when it is time to remove. We recommend using FreeDerm adhesive remover to remove CathGrip. You can also use warm water for removal by soaking CathGrip or wrapping a washcloth around the device until the hydrocolloid turns milky white and rolls gently from the skin.

### To Remove with FreeDerm®:

Wipe or spray at the edge of the adhesive, lifting away from the skin as it releases. Continue until entire adhesive has been removed. Depending on the size of the adhesive, this may require more than one spray or wipe.



Sold separately in a box of 30 wipes (SKU: BOM90037WBX). Also available as 1 oz. pump spray (SKU: BOM52200) and 3 oz. pump spray (BOM52202EA)


# Application Instructions

For Best Results  
Follow the 5 P's

- Prep • Peel • Press
- Place • Pull


1

## Preparation

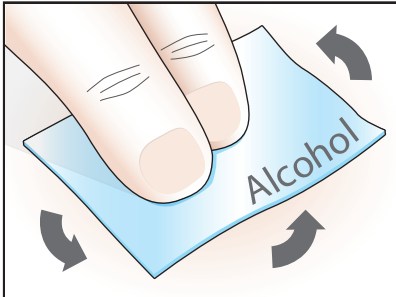


### A Wipe & Dry

Clean entire application site with alcohol wipe and/ or non-moisturizing soap. Allow to dry.

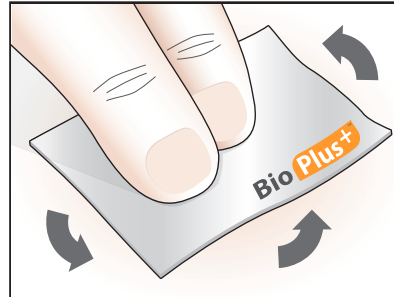

### B Wipe & Dry


Wipe entire application site with BioPlus+™. Allow 15+ seconds to dry.

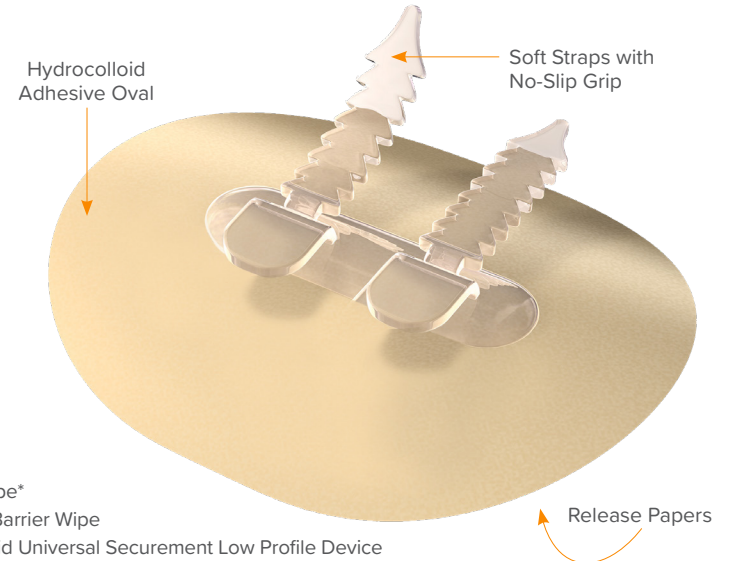

### Contents:


- 1 Alcohol Wipe\*
- 1 BioPlus+™ Barrier Wipe
- 1 Hydrocolloid Universal Securement Low Profile Device

\*Not included in part numbers ending with NA

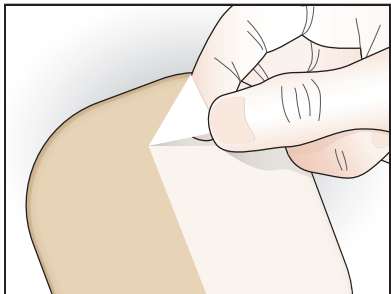


### 2 Peel

Remove the release papers from the hydrocolloid adhesive oval.

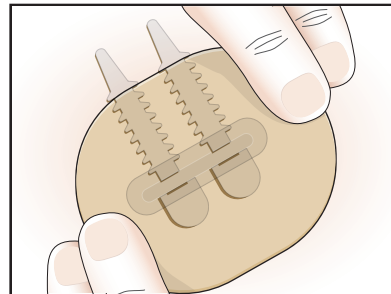


### 3 Press

Press the hydrocolloid oval to the skin for 15 seconds to seal.

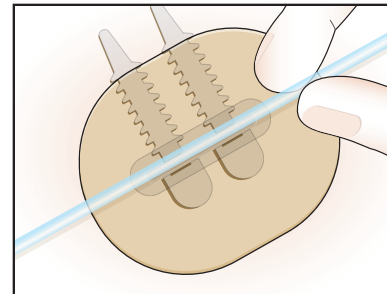

### 4 Place

Place the tube on the soft, no-slip grips.

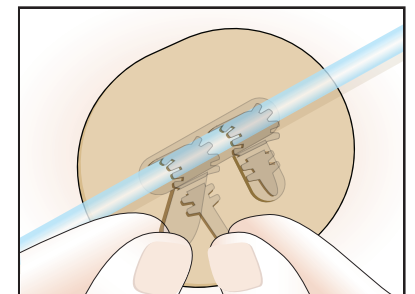

### 5 Pull & Secure

Pull the straps tight to secure.

**ATTENTION:** Following application, leave a one inch space around the CathGrip® device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.