

FreeDerm® Adhesive Remover




Instantly removes hydrocolloid without irritation and evaporates rapidly with no residue. Allows immediate re-application of an adhesive device without washing. Available as 1oz (30mL), 3oz (90mL) pump spray or single use wipes (6mL).

- ✓ Lotion-Free
- ✓ Unscented
- ✓ Oil-Free
- ✓ Alcohol-Free
- ✓ No Residue
- ✓ No Harsh Chemicals
- ✓ Rapid Dry Time

Benefits

- Efficiently removes adhesives including hydrocolloids, transparent dressings and tapes
- Minimizes the risks associated with Medical Adhesive Related Skin Injuries (MARSI)
- Quick release of adhesive = saves nursing time
- Minimizes patient discomfort and reduces pain
- Protects fragile skin while removing adhesives of all kinds
- **ALSO** can be used to remove sticky residue from equipment, bed rails, carts and much more...

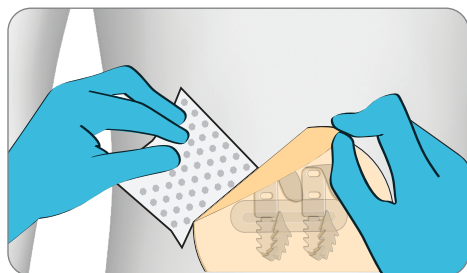

SKU#	SIZE
1oz Spray Bottle	52200
3oz Spray Bottle	52202
6 ML Wipes, Box of 30	90037

Instructions

For external use only, on normal intact skin

1. Spray or apply FreeDerm to the edges of the adhesive until wet.
2. Gently lift the adhesive edge from the skin. While lifting dressing/adhesive, spray or apply FreeDerm underneath the dressing to the exposed adhesive until completely released.

If redness or irritation occurs, discontinue use and consult your healthcare professional.

To place an order, please email Purchase Order to CustomerCare@BioDermInc.com



©2022 BioDerm, Inc. All rights reserved. BioDerm® and FreeDerm® are registered trademarks of BioDerm, Inc.

LT 94024 C